
## **Assessment of test results Sensitest Corona Covid-19 Rapid Test**



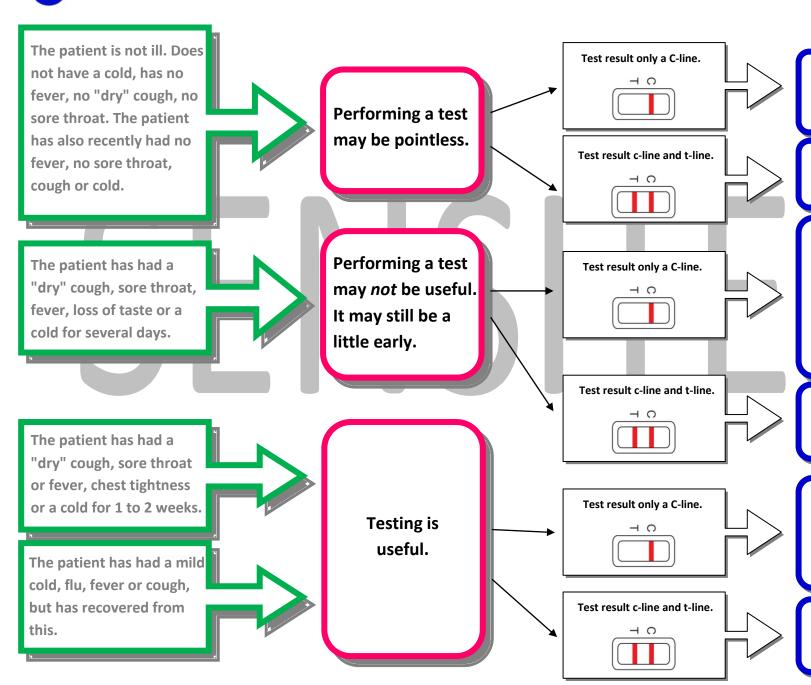

The c-line indicates that the test has been performed correctly. If no t-line appears, the patient is likely not in possession of Corona IgM or IgG antibodies.

The patient may have come into undetected contact with the Corona virus in the past and is now in possession of antibodies (IgG).

The c-line indicates that the test has been performed correctly. If no t-line appears, the patient may not have Corona IgM or IgG antibodies. Or you have tested too early and the patient has not yet produced enough IgM antibodies: test again within a few days. Complaints can also be caused by a seasonal flu or a cold other than the Corona virus.

The patient has probably recently come into contact with the Corona virus and now has the antibodies (IgM).

The c-line indicates that the test has been performed correctly. If no t-line appears, the patient is probably not in possession of Corona IgM or IgG antibodies. The patient may have (had) a seasonal flu. Or it is necessary to test again after a few days.

The patient has probably come into contact with the Corona virus and now has antibodies (IgM or IgG or both).